

## The Application

These mini luxators have an anatomically curved blade to adapt to the tooth root. Through a slight rotating movement the periodontal ligaments can be cut cleanly, at the same time dilating while preserving the alveolar bone.



### mesial



Position the luxator tip "directly" into the periodontal space on the mesial side of the root.

Insert the luxator into the periodontium until 2/3 of the root length with small axial movements and under slight pressure.



### distal



If the root remains fixed, repeat the same operation on the distal side. After this mesial and distal luxation the tooth should be dislocated.

The newly designed handle is small and light, allowing an easy grip and making adaption to the root easier and more effective.



The size of the handle allows both a comfortable handling similar to a pen and the classic, supported elevator position, even when the instrument is always kept axially.

## Our Suggestions


4 luxators of your choice, manageable packed in our p.i.c.®-tray type A.



1

Set 1  
Art. Nr. 1280.01  
with slender working ends

content:  
1271.00  
1272.00  
1273.00  
1274.00  
tray type A



2


Set 2  
Art. Nr. 1280.02  
with broad working ends

content:  
1273.00  
1275.00  
1276.00  
1277.00  
tray type A

**Your favorite set is not shown here?  
No problem!  
Assemble your favorite tray and save money!**


## The Different Types

### mesial


**1271.00** 2,5 mm I 

**1275.00** 5 mm I 

### distal

**1272.00** 2,5 mm I 


**1276.00** 5 mm I 

**1274.00** 2,5 mm I 

**1277.00** 5 mm I 

**1273.00** 4 mm I 

**1278.00** 2,5 mm I 

**1279.00** 2,5 mm I 



**For ordering or information please  
contact your local distributor!**